



THE MOST COMPACT CRYOSURGERY DEVICE

CryoPen sets a new standard for the cryogenic treatment of benign skin lesions, offering the ultimate in control, convenience, and cost-effectiveness – as well as an outstanding patient experience.

This high-performance, ergonomic cryotherapy pen dispenses a fine jet of nitrous oxide under high pressure, allowing you to work with millimeter precision.

The CryoPen XPS is our most compact model, and is suitable for intensive and varied use.


Key features

- Ergonomic design for superior control
- Leading-edge microtechnology for controlled gas flow
- Takes easy-to-load 8g N₂O cartridges
- Aluminum body with applicators in medical-grade materials
- Provided with 2 autoclavable applicators
- Suitable for intensive and varied use
- Certified **CE1639** **FDA** **MPSAP**






APPLICATORS

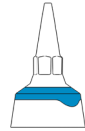

Ø 1-3mm
Power: ■

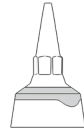

Ø 2-4mm
Power: ■ ■

Continuous flow time with an 8g cartridge



PRODUCT INFORMATION

Device

The CryoPen XPS is suitable for all treatment situations where delicate freezing power is required.

Procedure

No special preparation is required. Apply the jet of nitrous oxide for 2 to 45 seconds on each lesion, depending on the size and depth of the lesion. For larger lesions, use circular or back-and-forth movements to treat all the tissue.

What kind of lesions?

Acrochordon, actinic keratosis and cheilitis, capillary hemangioma, cutaneous leishmaniasis, discoid lupus erythematosus, filiform warts, flat warts, vulgar warts, plantar warts, anogenital warts, rosacea, lichen sclerosus, granuloma annulare, molluscum contagiosum, seborrheic keratosis and viral lesions.



CONTENTS OF THE STARTER SET

Device

CryoPen XPS

Applicators

- 1-3mm applicator
- 2-4mm applicator

Cartridges

12x 8g cartridges

Accessories

- Metal carrying case
- Forceps
- Device stand

Documentation

- Treatment overview
- User manual

BASICS

Dimensions

- Device: 20 x 2.75 x 2.75 cm
- Carrying case: 28.8 x 22.8 x 6.7 cm
- Starter set: 38 x 30 x 7.5 cm

HS codes

- 90189084 (device)
- 28112930 (cartridges)

Weight

- Device with a full cartridge: 200g
- Starter set: 2,356g

