

MELAG



DERMAT
MEDICAL SUPPLIES

016/ 40 05 45
info@dermat.be
www.dermat.be



MADE IN
GERMANY



Podo

SteriHero

My autoclave. My **superpower**.

✓ Now available as a medical device

Your new **superpower**

No matter what big tasks the day holds in store for you, **SteriHero Podo** has your back. The brand new stand-alone Class B and S autoclave now complies with medical device regulations and is perfectly suited for podiatry: From the features to the price! Trendsetting technologies, an incredibly advanced display and the unique MELAG quality - made in Germany give you the superpower to change everything:

Now also available as
**Class B & S
autoclave**



Supersafe

Safety for your patients: Class B and Class S autoclaves with the fractional vacuum procedure provide the safest sterilization of all wrapped and unwrapped instruments - regardless of their type and complexity.

Class B and S steam sterilization according to EN 13060

Superstrong

More than enough power for whatever you plan to do: High loading capacities, fast cycle times from 13 minutes and effective vacuum drying ensure that your instruments are ready for use in no time.

Up to 5 kg on 5 trays



Supersmart

The intuitive user interface with a large Smart-Touch Display makes everything easier: From program selection to documentation. With the help of process monitoring, you can provide complete proof of a successful sterilization.

Documentation via USB, network or log printout*

Superrobust

An autoclave should never fail: We make no compromises when it comes to reliability and develop under the highest quality requirements at our headquarters in Berlin.

100 % quality - made in Germany



MADE IN GERMANY



* Optional with MELAprint® 80

A dream team for your practice hygiene:



*„Our patients are our top priority:
With the system solution tailored to my
practice's needs, we offer maximum protection
with minimum effort.“*

Practice for podiatry, Vera Wille



*„We live up to our responsibility to use only
impeccable instruments to patients with our new
reprocessing workflow with solutions from MELAG.“*

Practice for podiatry, Reni Otte

Protects the environment. And your patients.

SteriHero Podo guarantees safe sterilization according to your needs with a wide range of programs. But that's not all: the pioneering stand-alone device concept and the Power Save energy-saving function make **SteriHero Podo** the cleanest solution - for your budget and the environment.

Programs SteriHero Podo 18	Universal-B	Quick-S	Quick-B	Gentle-B	Prion-B
Program specification	134 °C, holding time: 5.5 min	134 °C, holding time: 3.5 min	134 °C, holding time: 5.5 min	121 °C, holding time: 20.5 min	134 °C, holding time: 20.5 min
Operating times (at minimum to maximum load)*	22 – 27 min	15 – 18 min	22 – 25 min	37 – 42 min	37 – 42 min
Drying (time controlled)	20 min	5 min	10 min	20 min	20 min

Programs SteriHero Podo 18 S	Universal-S	Quick-S	Gentle-S	Prion-S
Program specification	134 °C, holding time: 5.5 min	134 °C, holding time: 3.5 min	121 °C, holding time: 20.5 min	134 °C, holding time: 20.5 min
Operating times (at minimum to maximum load)*	18 – 23 min	13 – 19 min	33 – 37 min	33 – 38 min
Drying (time controlled)	25 min	5 min	25 min	25 min

* Data for the Quick S program for unpacked loads and for all other programs for single-packed loads. Times may vary depending on load type and quantity.

Device / Order no.	Device type	Chamber size	Chamber volume	Load quantity	Device dimensions (W x H x D)	Weight	Power supply	Documentation interfaces
SteriHero Podo 18 ME10118PB	Stand-alone class B autoclave (compatible with additional water treatment)	ø 25 cm x 35 cm long	17 liters	max. 5 kg instruments / 1.8 kg textiles	47 x 50 x 64 cm (device fits on a 60 cm table top)	48 kg	220 – 230 V / 50 – 60 Hz (2.100 Watt)	USB, Network
SteriHero Podo 18 S ME10118PS	Stand-alone class S autoclave (compatible with additional water treatment)	ø 25 cm x 35 cm long	17 liters	max. 4 kg instruments / 1 kg textiles	47 x 50 x 64 cm (device fits on a 60 cm table top)	46 kg	220 – 230 V / 50 – 60 Hz (2.100 Watt)	USB, Network

Rev.0-24/0039-01.24-EN-ME



Save paper, protect the environment:

In order to reduce paper, we provide you with electronic user documentation via our Download Center. Further information at: www.sterihero.com

MELAG