

Dermojet polymedical

PRODUCT REFERENCE: 475



Removable tips of the Dermojet Polymédical

The Dermojet Polymédical comes with 2 removable tips that can be adapted to the models references 500, 510 and 520.

These high quality stainless steel tips have the advantage that they can be separated from the body of the device and allow for more precise aiming (tip diameter: 0.9 cm, 1.7 cm for the ring of the HR model).

Dosage: Single dose of 0.1 ml or 0.05 ml.

This hand-held instrument is cylindrical in shape and consists of 3 parts:

1. The body of the syringe contains a plunger, a tension spring and a tension lever.
2. A 4 ml Pyrex or Radel® reservoir
3. An injection head equipped with a "Tip Jet" attachment system

Operation

- Insertion of the solution to be injected into the tank
- Arming the injector by lowering the lever
- Release of the liquid by pressing the release button

Weight: approx. 250 g

Applications

- Local anaesthesia,
- Injection of corticoids,
- Medical treatment of hypertrophic and keloidal scars
- Treatment of warts
- The Polymedical Dermojet works without cartridge, bottle or ampoule
- The Polymedical Dermojet does not contain any medicinal substances.

It is the practitioner who, when filling the tank, chooses the solution to be injected.